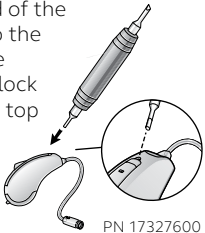
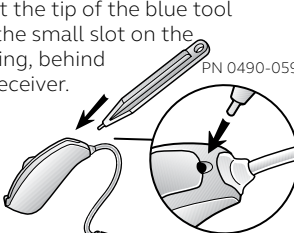

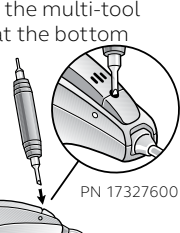
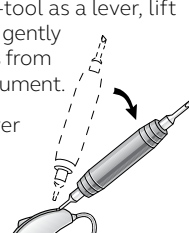
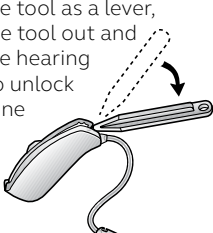
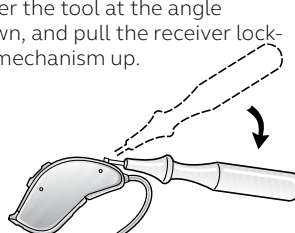
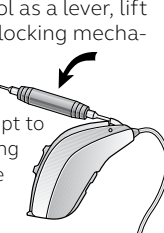
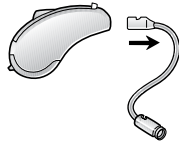
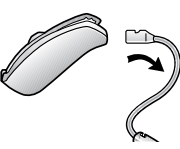
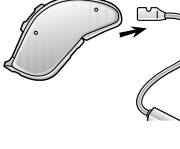
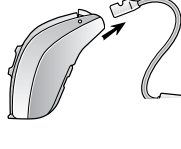
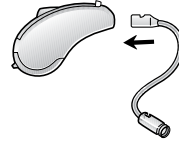
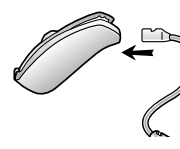
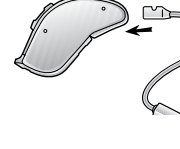
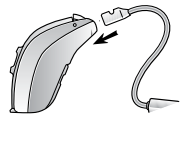
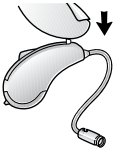
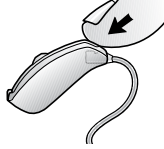
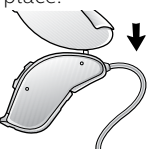
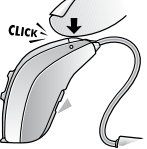


# Changing a Sure Fit receiver

ReSound LiNX 3D™ Rechargeable <b>61-DRWZ</b>	ReSound LiNX 3D™ and ReSound LiNX2™ <b>61</b>	ReSound LiNX 3D™ and ReSound LiNX2™ <b>62</b>	ReSound Enya™ <b>62</b>
<p>Insert one end of the multi-tool into the opening of the clear receiver lock just above the top microphone.</p>  <p>PN 17327600</p>	<p>Insert the tip of the blue tool into the small slot on the housing, behind the receiver.</p>  <p>PN 0490-059</p>	<p>Insert the tip of the white tool into the small slot in the receiver locking mechanism.</p>  <p>PN 15502500</p>	<p>Insert one end of the multi-tool into the cut-out at the bottom of the locking mechanism.</p>  <p>PN 17327600</p>
<p>Using the multi-tool as a lever, lift the receiver lock gently until it protrudes from the hearing instrument.</p> <p>Note: the receiver lock does not need to be completely removed.</p> 	<p>Using the blue tool as a lever, swing the blue tool out and away from the hearing instrument to unlock the microphone cover.</p> 	<p>Lower the tool at the angle shown, and pull the receiver locking mechanism up.</p> 	<p>Using the multi-tool as a lever, lift the bottom of the locking mechanism to unlock the receiver tube.</p> <p>NOTE: Do not attempt to disconnect the locking mechanism from the hearing instrument.</p> 
<p>Pull the receiver out to remove it from the hearing instrument.</p> 	<p>Pull the receiver down &amp; out to remove it from the hearing instrument.</p> 	<p>Pull the old receiver tube straight out to remove it from the hearing instrument.</p> 	<p>Pull the old receiver tube straight out to remove it from the hearing instrument.</p> 
<p>To insert a new receiver, push the new receiver tube until it is fully seated.</p> 	<p>To insert a new receiver, push the new receiver tube in until it is fully seated.</p> 	<p>To insert a new receiver, push the new receiver tube in until it is fully seated.</p> 	<p>To insert a new receiver, push the new receiver tube in until it is fully seated.</p> 
<p>Lock the receiver tube in place by pushing the clear receiver lock into the hearing instrument until it is flush.</p> 	<p>Lock the receiver tube in place by clicking the locking mechanism closed.</p> 	<p>Lock the receiver tube in place by firmly pushing down the locking mechanism until snugly in place.</p> 	<p>Lock the receiver tube in place by clicking the locking mechanism closed.</p> 
<p>Attach a dome or custom Micromold to the receiver.</p>	<p>Attach a dome or custom Micromold to the receiver.</p>	<p>Attach a dome or custom Micromold to the receiver.</p>	<p>Attach a dome or custom Micromold to the receiver.</p>

## Manufacturer according to FDA:

### ReSound North America

8001 E. Bloomington Freeway  
Bloomington, MN 55420  
1-800-248-4327  
resoundpro.com

### ReSound Government Services

8001 E. Bloomington Freeway  
Bloomington, MN 55420  
1-800-392-9932  
resound.com/governmentservices

## Manufacturer according to Health Canada:

### ReSound Canada

303 Supertest Road  
Toronto, Ontario M3J 2M4  
1-888-737-6863  
resoundpro.com

